C | S

47101  
MITELLA  
COTTON

47101

**ARMSLINGA BOMULLSVÄV**

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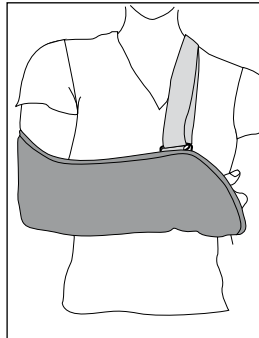
## 47101 MITELLA COTTON

Carefully read this instruction sheet before use!

# ENGLISH

**Intended use/Indications:** Intended for temporary off-loading for shoulder and arm. Indications: Post-trauma and post-surgery conditions.

**Contraindications:** None known.

**Features:**

The shoulder strap can be adjusted by releasing the Velcro strap and tightening or loosening. The two D-rings allow the sling to open wider for use with a cast. Integrated into this sling is a thumb loop to keep arm inside sling.

**Adjustments:**

This product should only be altered by a medical professional.

**Washing Instruction:**

Close all Velcro prior to laundering. Machine wash in 60°C (140°F). Dry flat. Do not tumble dry or dry clean.

**Material Specifications:**

Main material: 50% Cotton, 50% Polyester  
Velcro: 100% Nylon  
Thumb/Shoulder strap and Edging Tape: 100% Polyester.  
D-ring: 100% POM  
This product is not flame retardant.

**Who to Contact:**

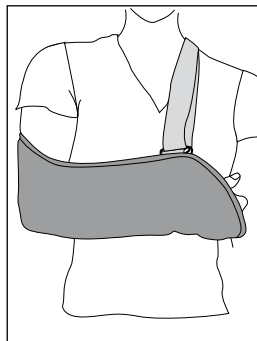
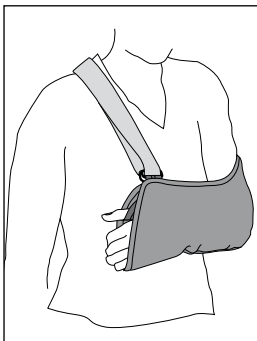
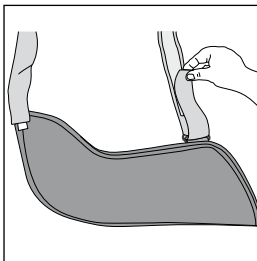
If any problems occur, remove the support and contact your fitter or medical professional.

**Time of use:**

Follow your medical professionals guidelines for use and limitations. This product is recommended for single person use only.

## Instructions for use

1. Check that the product is complete and without damage.
2. Position the arm into the sling and place the strap over head so the strap comfortably rests on the opposite shoulder.
3. Adjust the strap length so the elbow angle is approx. 90° .
4. Adjust for comfort.

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Camp Scandinavia AB, Karbingatan 38