# BASIC SHORT & LONG

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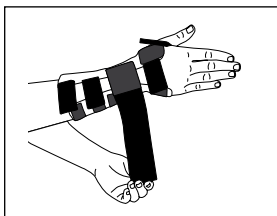
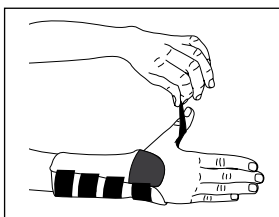
Carefully read this instruction sheet before use!

### Indications / Intended Use

These products is intended for instability in the wrist in combination inflammations, work overload, joint stress and rheumatic problems. The longer version is also suitable for post-fracture treatment.

### Contraindications

No known.

### Adjustments

This product should only be altered by a medical professional person.

### Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check you skin on a daily basis.

### Material Specifications

This product is made from: 74% Cotton, 19% Nylon and 7% Lycra with an aluminium support. Shape the aluminium support carefully to fit the shape of the thumb. Avoid repetitive bending. This product does not contain Latex, and is not flame retardant.

### Washing Instruction

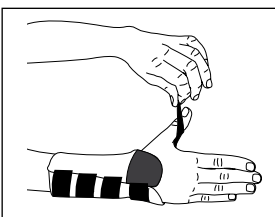
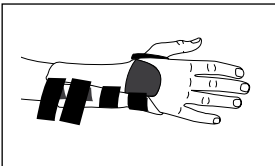
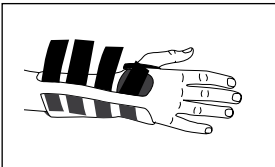
Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

### Who to Contact


If any problems occur, remove the support and contact your fitter or store where purchased.

---

## Instruction for use

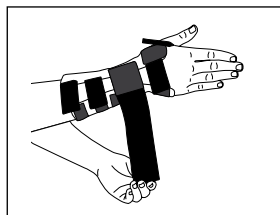1. Check that the support is complete and without damage.
  2. Open all Velcro straps.
  3. Place the hand over the support with the inside facing the aluminium spoon.
  4. Secure all Velcro straps. Adjust the thumb strap.
  5. For the long model, secure the extra circular strap around the wrist.
- Adjust for comfort.

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