



Instructions for Use






Selection®
Ulnar Deviation
Art. No. 35218

Instructions For Use

EN	SELECTION® ULNAR DEVIATION	p. 6
SV	SELECTION® ULNAR DEVIATION	p. 7
DA	SELECTION® ULNAR DEVIATION	p. 8
NO	SELECTION® ULNAR DEVIATION	p. 9
FI	SELECTION® ULNAR DEVIATION	p. 10
DE	SELECTION® ULNAR DEVIATION	p. 11
NL	SELECTION® ULNAR DEVIATION	p. 12
IT	SELECTION® ULNAR DEVIATION	p. 13
FR	SELECTION® ULNAR DEVIATION	p. 14
ES	SELECTION® ULNAR DEVIATION	p. 15
JA	SELECTION® ULNAR DEVIATION	p. 16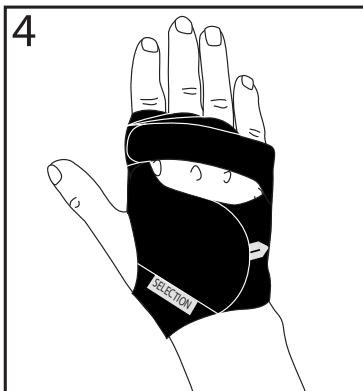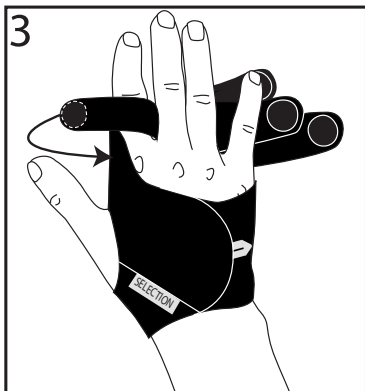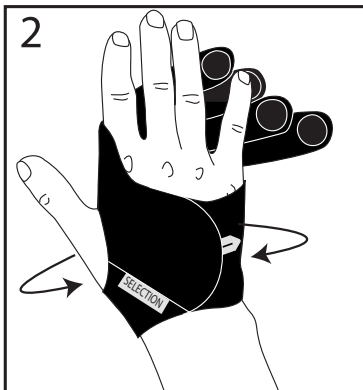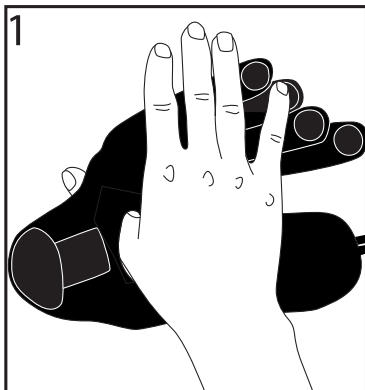

5

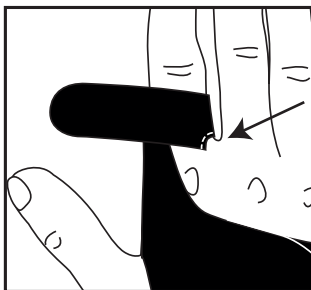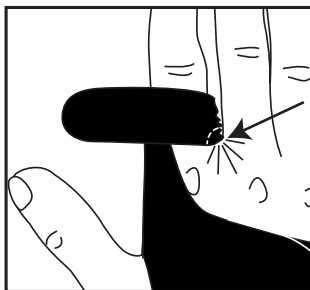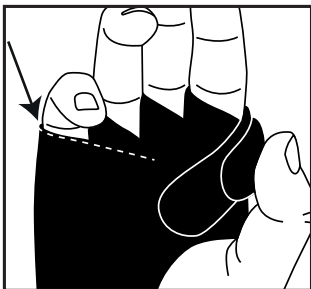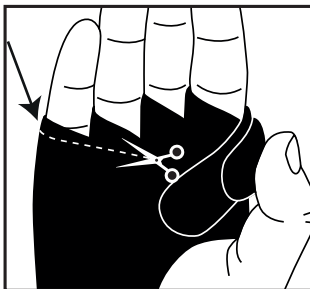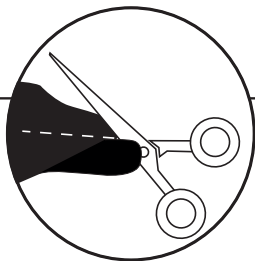EN 35218 Selection® Ulnar Deviation

Indications / Intended Use

Textile hand/finger-orthosis for management of Ulnar Deviation. Corrects alignment of joints of the fingers to improve extension, strength and function of the hand and thereby can provide reduced pain and joint stiffness. For patients with Arthritis. Ulnar deviation is likely to be a characteristic for those with various forms of inflammatory diseases, such as Rheumatoid arthritis. Can also be used for realignment and stabilization in people with joint laxity and muscle weakness. Can be combined with a Selection Wrist orthosis.
Single Patient-multiple use.

Contraindications

Fixed contractures.

Adjustments

This product should only be altered by a medical professional. Minor adjustments can be done during fitting. It is possible to trim the material without fraying.

Warnings

Ensure that the straps are not too tight. Do not use when the product is wet or on wet skin. Check your skin daily for irritation.

At first sight of skin irritation remove the orthosis and contact your treating clinician.

Material Specifications

This orthosis is made from nylon, polyester, lycra. This product is not flame retardant. Latex free.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Disposal

Disposal in accordance with local legislation

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use (See illustrations on page 4-5)

Check that the product is intact and free from damage. Check that you have the correct size and side (L/R).

1. Insert the thumb through the opening.
2. Close the Velcro Strap over the back of the hand.
3. Fasten the strap around the index finger first, then continue with the rest of the fingers.
4. Adjust the straps until the fingers are correctly positioned.
5. If necessary, the orthosis can be customised by trimming with scissors.

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