28042
BISCA WRIST

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Intended Use

This product is intended for patients with rheumatic problems. It is lined with a soft cotton material that is very comfortable towards the skin.

The integrated strap combined with the aluminium stay gives a good support.

Indications

Rheumatic illnesses and for pain relief.

Contraindications

No known.

Adjustments

This product should only be altered by a medical professional.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This orthosis is made from cotton, elastane and with a filling of 100% polyurethane. This product is not flame retardant. Latex free.



Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instruction for use

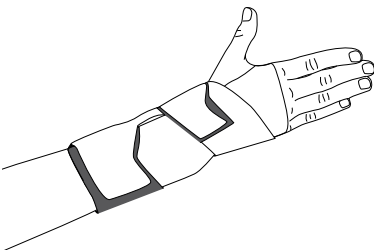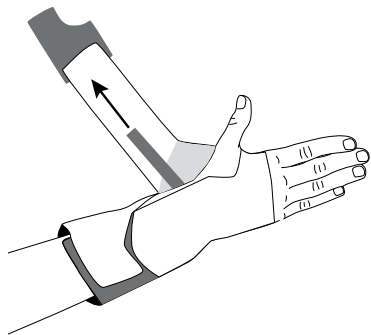1. Check that the product is complete and without damage.
2. Open all Velcro straps.
3. Put the hand through the large opening and the thumb through the smaller.
4. Start by attaching the small Velcro strap.
5. Then close the long strap around the wrist. Ensure it is not too tight. Should have a comfortable fit.
6. Adjust for comfort.

Item No.	Color	Size	Side	Wrist circumference
28042 1011	Blue	Small	Left	11 - 14 cm
28042 1012	Blue	Medium	Left	14 - 17 cm
28042 1013	Blue	Large	Left	17 - 20 cm
28042 2011	Blue	Small	Right	11 - 14 cm
28042 2012	Blue	Medium	Right	14 - 17 cm
28042 2013	Blue	Large	Right	17 - 20 cm