C | S

47500  
KNEE  
IMMOBILIZER

# SVENSKA

## 47500 KNÄIMMOBILISERINGSSKENA

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## 47500 KNEE IMMOBILIZER

Carefully read this instruction sheet before use!

### ENGLISH

**Intended use/Indications:** Intended for temporary immobilizing of the knee joint. Can also be used for support while standing. Indications: Post-trauma and postoperative conditions.

**Contraindications:** None known.

**Features:**

This product consists of 3 parts. The two side panels should be adjusted for optimum sizing. The dorsal stays can be gently shaped into flexion. Adjustments of the side panels and shaping of the stays should only be performed by a medical professional.

**Adjustments:**

This product should only be altered by a medical professional.

**Washing Instruction:**

Close all Velcro prior to laundering. Remove Aluminum stays before wash. Machine wash in 60°C (140°F). Dry flat. Do not tumble dry or dry clean.

**Material Specifications:**

Main material: Surface 100% polyester, foam made from polyurethane.

Velcro: 100% nylon

D-ring: 100% polypropylene

Tape/Edging: 100% polyester.

Stays: Aluminum

This product is not flame retardant.

**Who to Contact:**




If any problems occur, remove the support and contact your fitter or medical professional.

**Time of use:**

Follow your medical professionals' guidelines for use and limitations. This product is recommended for single person use only.

## Instructions for use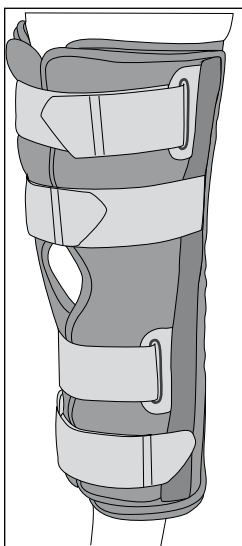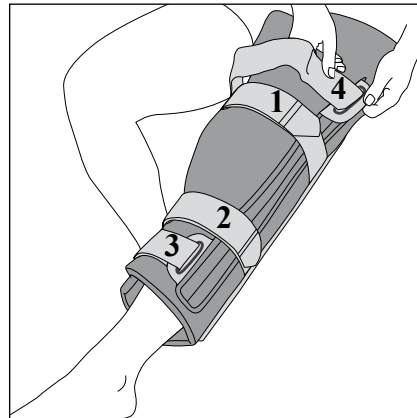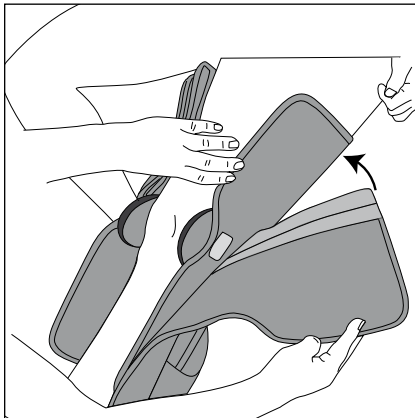

1. Check that the product is complete and without damage.
2. Position the large panel around the leg. Loosen the removable panels and adjust for correct size. This adjustment should only be done by the fitter during initial fitting.
3. At initial fitting the stays in the back of the brace can be slightly shaped. This should only be done by the fitter.
4. Attach the 4 straps. Start with the strap just above the knee, then attach the strap below the knee.
5. Adjust all straps for comfort, ensuring the fit is snug enough to avoid downward migration of the brace.

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Camp Scandinavia AB, Karbingatan 38